# K-Active®

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**EN** K-Active Tape is an elastic adhesive tape which is made out of cotton fabric and an acrylic adhesive coating that is activated by rubbing after application. K-Active Tape uses a low irritant acrylic adhesive. Because of the permeability of the tape to air and liquids and the water-repellent version, the dressing can be worn for up to several days. Superior breathability is achieved with NITTO DENKO's unique wave pattern adhesive coating technique, to avoid skin irritation caused by sweat or maceration. The use of extensible material means that the body is not restricted in terms of its normal physiological movements.

Apply K-Active Tape only after consulting your doctor or your K-Active Therapist and in accordance with K-Active Therapy.

### Application:

- If a lot of hair is present, shave the area first, then clean and dry the skin thoroughly before applying the tape. Do not apply K-Active Tape to damaged or irritated skin.
- K-Active Tape is applied unstretched to pre-extended areas of muscle or skin parts. If special methods are used, the tape can be stretched before application.
- Remove K-Active Tape with care in the direction of hair growth.

### Warnings:

- If you have sensitive skin, you should test a small piece of K-Active Tape on the skin first. Should this cause irritation, no more tape should be applied.
- In rare cases, this tape may cause rashes. However, if symptoms such as itching and irritation appear, remove the tape immediately.
- If the dressing is used on stretched muscle or skin, irritations may appear.
- Shower/take a bath as usual and dab off any remaining dampness carefully with a towel.
- Keep out of reach of children
- Store at 5 to 30 °C and away from direct sunlight
- For single use only

### Intended use:

- K-Active Tape protects and supports muscles and is not for securing joints in place.
- It limits the range of muscle extensibility to prevent muscle strain and injuries or to alleviate or avoid pain.
- K-Active Tape PreCut is pre-cut into shapes for each purpose and does not need to be cut before application.

### Where to apply:

- PreCut-X: Elbows, back (shoulder blades)
- PreCut-Y: Shoulders, hips, knees, calves, hamstrings
- PreCut-Lymphatic: Soles, thighs, shoulders, knees

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